





UTZirc 100% Ultra Translucent

# Perfect Balance of Beauty and Strength

Made from a unique formula of Ultra Translucent Zirconia, UTZirc 100% restorations are proudly created in our Australian facilities by our local technicians. Its ultra translucent properties allow you to achieve predictable, natural and more aesthetic results while also maintaining strength.

UTZirc 100% Ultra Translucent is an ideal aesthetic solution for anterior crown and bridge cases of up to three units as well as single unit posteriors. Manual staining and glazing is also applied to enhance the final aesthetic results.

## Translucency

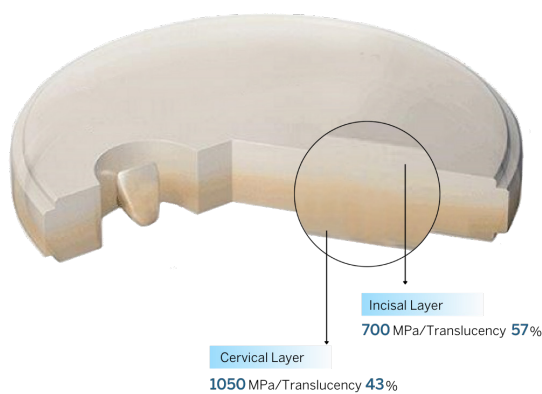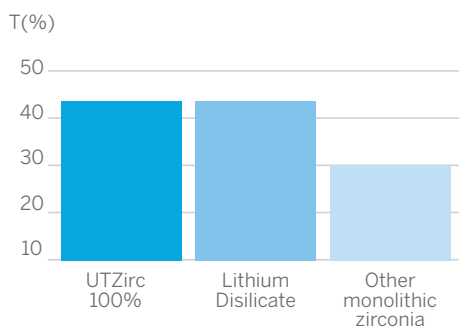

# Ultra Translucent UTZirc 100% Preparation Guide

**10 Year**  
warranty

**CAD/CAM**  
restorations

## Cementation Guide

Self-adhesive or adhesive resin cements are recommended. The shade of the cement will show through and will contribute to the final shade.

## Adjustment Guide

Fine diamond with water cooling or a diamond impregnated rubber. Areas must be polished to a high shine with diamond polishing paste afterwards.

## Stump Shade

The shade of the stump should be provided as any discolorations may affect the final shade of the UTZirc 100% crown.

Material	Ultra Translucent Monolithic Zirconia
Translucency	✓ ✓ ✓ ✓ ✓
Flexural Strength	700MPa incisal and 1050MPa cervical
Aesthetics	✓ ✓ ✓ ✓ ✓
Durability	✓ ✓ ✓ ✓ ✓
Margins	Shoulder/chamfer
Indications	Anterior and Posterior Crowns, Inlays/Onlay, Bridges up to Full Arch, Cement-Retained/Screw-Retained Implants
Cementation	Conventional or Adhesive
Impression technique	Digital or traditional
Turnaround time	7 in-lab days. Also available in the 48 hour range

## Preparation Guide

